


BBID Banding and Labeling Requirements

Initial collection (no BBID Band)

1. Place ADT label that printed with specimen label on the BBID wristband

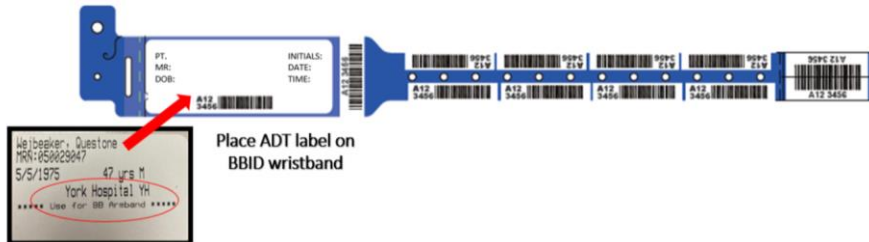

2. Place band securely on patient's wrist.



3. Apply barcode specimen label to the tube AFTER collection



4. Add a BBID sticker to tube.

Peel off a label from BBID and place it length wise on pink top



If BBID band exists

1. Check location on BBID band



- a. If location is your current associated facility – continue to STEP #2 in **this column**.
- b. If different – CUT off the BBID band and start a new collection following the Initial Collection Instructions (Left Column)

2. Apply barcode specimen label to the tube AFTER collection



3. Add a BBID sticker to tube.

Peel off a label from BBID and place it length wise on pink top



When is Scanning Required vs CERT FORM required

<p>Patient Armband AND Specimen Label scan are <u>REQUIRED</u></p>	<p>If the collection activity shows the Redraw Later button or the Accept button, patient and label scanning is <u>REQUIRED</u>.</p> <div data-bbox="447 289 1759 396"> </div> <p>--OR--</p> <div data-bbox="447 500 1596 607"> </div>
<p>WellSpan Health Blood Bank Sample Collection Certification Form (aka <u>CERT FORM</u>) is <u>REQUIRED</u>.</p>	<p>If the collection activity shows the Collect All button, a CERT FROM is <u>REQUIRED</u></p> <div data-bbox="447 711 1688 818"> </div> <p>--OR--</p> <p>If a collector enters an override reason in either of these two windows during a scan required collection session, a <u>CERT FORM</u> is <u>REQUIRED</u>.</p> <div data-bbox="447 1092 1646 1312"> </div> <p>--OR--</p> <p>It is ALWAYS recommended to scan specimen labels to document collection if a barcode scanner is available.</p>